1. Introduction

Ambu® Baby is an instruction and training manikin for the simulation of realistic and correct anatomical conditions during cardiopulmonary resuscitation by means of insufflation and external cardiac compression.

Like the Ambu Man, the Ambu Baby features a unique, patented hygienic system under which each trainee gets his own face piece and head bag, thereby minimising any risk of cross-infection between trainees.

With the special hygienic system, disassembly and internal cleaning and disinfection are rendered unnecessary both during and after training.

The Ambu Baby provides an exceptionally lifelike representation of the human anatomy, particularly of those features important to training in modern resuscitation techniques.

2. Operating principles - see ①

When air is blown through the mouth and nose of the face piece (1.1), the head bag (1.2) fills, thereby displacing the air in the skull (1.3) through the valve (1.4) which is open when the head is slightly hyperextended. The valve closes if there is no hyperextension or if the head is extended too far.

Air will then pass through the neck tube (1.5) down into the lung bag (1.6) which expands and elevates the chest.

If insufflation takes place too quickly and/or the air volume is too high, the stomach valve (1.7) will open and allow air to flow into the stomach bag (1.8), thereby distending the stomach region.

When external cardiac compression is performed, the chest is squeezed and an expanding pad (1.9) will cause the chest to refill.

External cardiac compression must be concluded before starting insufflation to minimise the risk of stomach inflation. This applies to Ambu Baby as well as to resuscitation of real babies.

By operating the on/off slider (1.10) on the side of Ambu Baby, the airway can be blocked to simulate airway obstruction irrespective of the position of the head.

It is possible to simulate a brachial pulse on the right upper arm by manually activating the attached bellows.

Arms and legs are permanently attached and cannot be removed.

3. Hygienic system

The head bag is placed in the skull and the face piece is fitted. Insufflation has just started (see ②).

Insufflation is concluded and expiration has just started (see ③).

Note: To make the hygienic system to function as intended, each trainee must use his own face piece and head bag during training.

4. Preparation for training

The Ambu Baby manikin is supplied in a soft, nylon carrying case. Unpack the manikin and place it in position for training, for example on a table.

Mounting the head bag and the face piece:

Roll both sides against each other or fold it as shown (see ④). Insert the head bag as shown (see ⑤).

Fold the edges sideways. Make sure that the neck of the head bag is bended over the bushing and opened so that air can be blown into the head (see ⑥ and ⑦).

Fit the face piece by fixing the edges into the slot between the skull and the ears. Put the face piece into position by pressing on the nose as shown on the photo (see ⑧).

5. Instruction

The Ambu Baby is designed to meet all the training recommendations of the American Heart Association (AHA).

In general, however, it is recommended that instruction should be based on the techniques and provisions applicable in the country in which instruction takes place.

Instruction may be given in correct head movement. The airway will be obstructed on incorrect position.

The illustration shows insufficient hyperextension of the head. The airway is closed (see ⑨).

Correct head position. The airway is open (see ⑩).

Excessive hyperextension. The airway is closed (see ⑪).

Simulated airway obstruction. The on/off slider is pushed towards the legs (see ⑫).

Airway obstruction not simulated. The on/off slider is pushed towards the head (see ⑬).

The brachial pulse can be felt on the right upper arm. It is activated when the instructor squeezes the bellows (see ⑭).

Correct tilting of the head during insufflation (see ⑮).

The thorax movement is clearly visible during insufflation and expiration. The expiratory air can be felt and heard. There is no risk of crossinfection since the hygienic system ensures that the expiratory air is the same as that insufflated by the trainee in the head bag. Accidental stomach inflation is simulated and the abdomen visibly distended (see ⑯).

External cardiac compression can be practised according to current standards (AHA) (see ⑰).

6. Removal of face piece and head bag

6.1 Face piece

Pull the mask upwards until free of the skull (see ⑯).

6.2 Head bag

Grip the bag on each side without closing the opening. Pull the bag slowly upwards, twisting it from side to side if necessary to ease removal (see ⑯).

7. Cleaning and disinfection

With the patented hygienic system of the Ambu Baby, incorporating exchangeable face piece and head bag, no internal cleaning and disinfection is necessary.

7.1 Head bag

Each trainee has his own disposable head bag which should always be discarded after training.

7.2 Face piece

The face pieces can be reused after cleaning and disinfection.

7.3 Manual washing

- Rinse the face pieces in clean water.
- Wash items in warm water, max. 65°C, using a mild detergent.
- Rinse thoroughly in clean water to remove all traces of detergent and allow to dry before storing.

7.4 Machine washing

The face pieces can be washed automatically. Apply normal amount of detergent and choose a washing programme with a maximum temperature of 65°C. Allow items to dry before storing.

7.5 Disinfection

After cleaning the face pieces, disinfection can be performed as follows:

- A. Place the items in a sodium hypochlorite solution with minimum 500 ppm freely accessible chlorine (1/4 of a cup of liquid household bleach per gallon (approx. 4 litres) of tap water for 10 minutes). This solution must be fresh and should be discarded after use.
- B. Place the items in a 70% ethylalcohol solution for minimum 30 minutes.
- C. Chemical disinfection can also be performed using recognised disinfectants suitable for use with polyvinylchloride (PVC). The supplier's directions on dosage and disinfection period should be closely followed.

Always rinse the face pieces in clean water after disinfection and allow to dry before storing.

7.6 Cleaning of skull, neck, body, arms and legs

The skull, neck, body, arms and legs of the Ambu Baby should be wiped over with a cloth moistened in a mild detergent and then wiped over with a cloth moistened in clean water.

The training manikin must not be submerged in water nor should water be allowed to enter the internal parts.

Marks on the Ambu Baby left by lipstick or ball pen can penetrate the material and should therefore be removed as quickly as possible using alcohol.

7.7 Cleaning of clothing

The baby panties are made of 100% cotton and are washable in the usual way for such material at max. 60°C.

Warning: The face pieces must not be boiled, autoclaved or gas sterilised.

8. Specifications

Weight including carrying case:	2.5 kg (5.5 lbs)
Weight excluding carrying case:	1.8 kg (4 lbs)
Total length of manikin:	40 cm (approx. 16 inches)

Materials

Parts

Head	PVC, hard
Skin (body)	PVC, soft
Face piece	PVC, soft
Head bag	Polyethylene
Arms and legs	PVC, soft

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