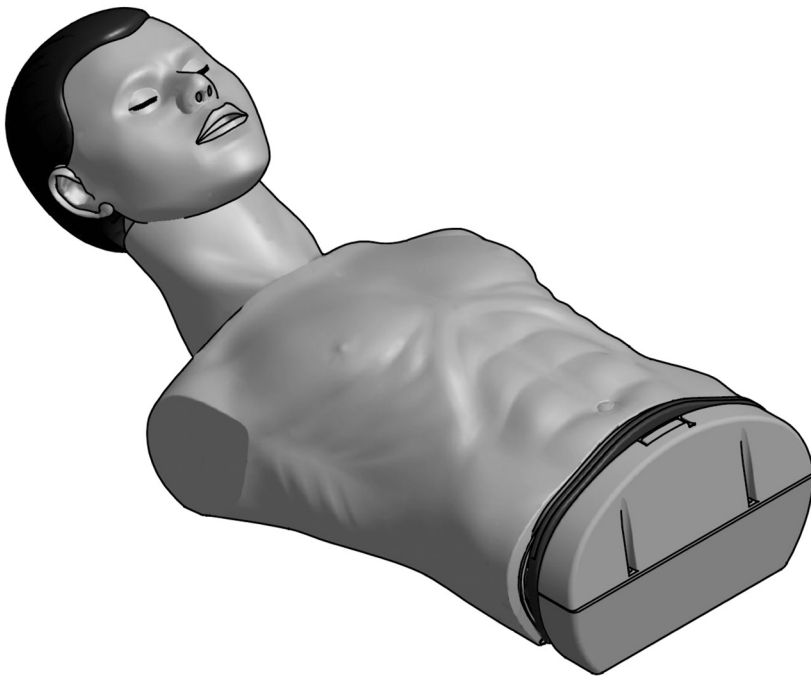
Ambu A/S
Baltorpbakken 13
DK-2750 Ballerup
Denmark
T +45 72 25 20 00
F +45 72 25 20 50
www.ambu.com

Directions for use

Ambu® AmbuMan Basic

English

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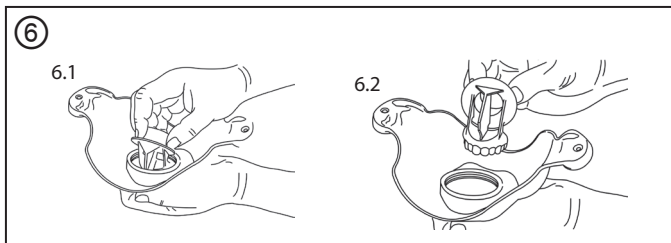
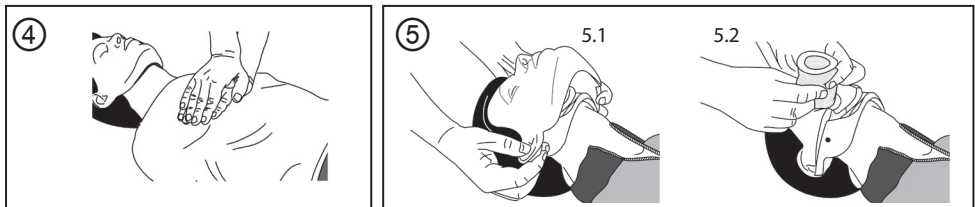
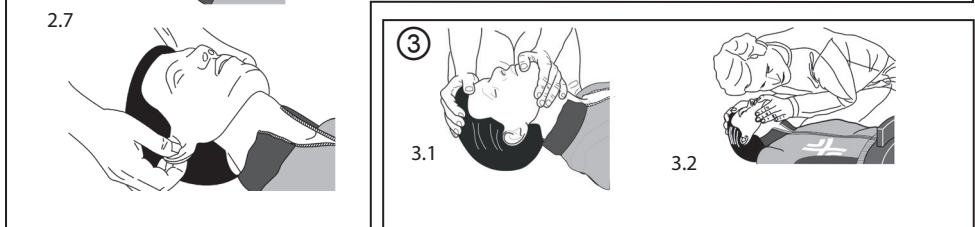
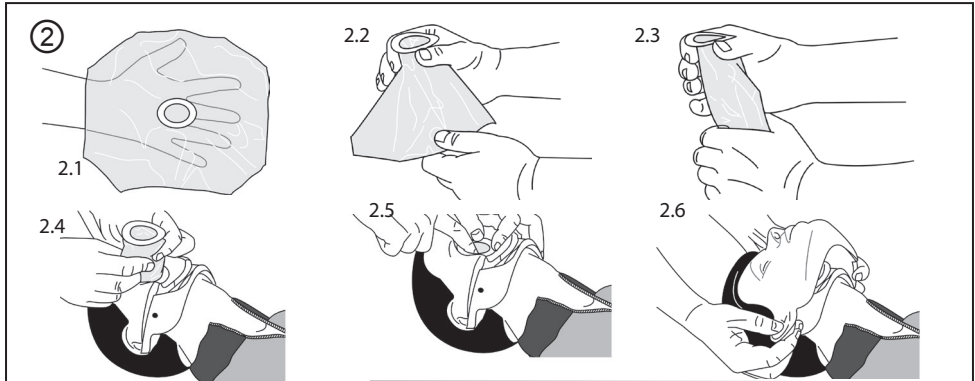
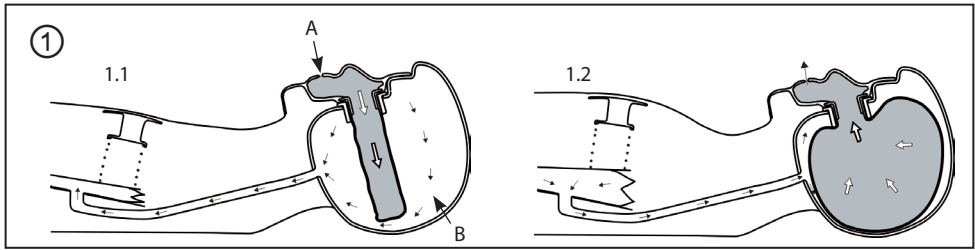
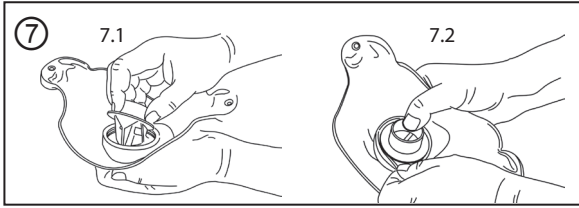
Content	Page
1. Introduction	4
2. Specifications	4
3. Functions	5
4. Preparation for training	5
5. Usage of the manikin	6
6. Cleaning and disinfection	6

Ambu® is a registered trademark of Ambu A/S, Denmark.

1. Introduction

The Ambu® Man Basic is an instruction and training manikin for the simulation of realistic conditions during cardiopulmonary resuscitation by means of rescue breathing and chest compression moreover the AmbuMan Basic will cover all aspect of basic life-support training.

The manikin incorporates a unique, patented hygienic system under which all trainees get their own face piece and head bag, thereby minimizing any risk of cross-infection between the trainees. The hygienic system also features realistic expiration from the manikin through the mouth and nose.

With the special hygienic system, disassembly and internal cleaning and disinfection are rendered unnecessary both during and after training. The manikin has the size of a normal adult and provides an exceptionally lifelike representation of the human anatomy, particularly of those features important to training in modern resuscitation techniques.

2. Specifications

Weight: 12 kg

Dimensions (L x W x H): 70 x 37 x 25 cm

Maximum chest compression: 7,5 cm

Part/Material

Basic unit	Polyethylene
Instrument part	ABS plastic
Skull	PVC, hard
Chest skin	PVC, soft
Face piece	PVC, soft
Head bag, disposable	Polyethylene
Training suit	50 % cotton and 50 % polyester
Carrying bag/training mat	nylon material

Usage and storage:

Storage temperature (non-condensing): -18 °C (-0,4 °F) to 40 °C (105 °F)

Usage temperature (non-condensing): -5 °C (23 °F) to 40 °C (105 °F)

Humidity: 5 % to 95 %

3. Functions

3.1 Hygienic System ①

Schematic diagram:

The head bag has been mounted in the skull and the face piece fitted.

Ventilation has just started, see (1.1).

A: Trainee's air B: Ambient air

Ventilation is concluded and expiration has just started, see (1.2).

NOTE: If a leakage occurs around the mask or if insufflation proves difficult, it is normally due to incorrect positioning when lifting the chin or to insufficient hyperextension of the head. To comply with the requirements of the hygienic system, and to achieve a proper seal and accurate measuring results, a head bag must always be used. All trainees must use their own face piece and head bag only during training.

3.2 Compression indicator

The compression indicator shows the correct depth of the chest compression. When compressing the chest of AmbuMan Basic the indicator will show a red bar until the correct depth is reached. Once the correct depth is reached the indicator will change into a green bar. If the compression is continued too deep the indicator will change back into a red bar.

The compression indicator will show a green bar at a compression depth of 5cm to 6 cm.

3.3 Ventilation indicator

The correct ventilation volume is indicated by the blue marker in the mechanical display.

4. Preparation for training

The AmbuMan Basic training manikin is supplied in a carry bag. To start the training remove the manikin from the bag.

4.1. Mounting the head bag and face piece ②

Unfold the head bag and hold as shown, see 2.1.

Holding the ring of the bag, fold together as shown on the photos, see 2.2 and 2.3.

Insert bag in the head, see 2.4.

Smooth out the edges along the opening in the skull, see 2.5.

Hold the face piece by the ears and position on the skull so that the top is flush with the hairline, see 2.6.

Pull face piece downwards by holding firmly by the ears until in position. Make sure that the edges of the mask are located under the hair at the temples. Fix the openings at the back of the ears on the fixing prongs on each side of the skull, see 2.7.

5. Usage of the manikin

5.1 Ventilation ③

The patented hygienic system of the Ambu enables the trainee to get a realistic feeling for

performing ventilation.

The following techniques can be used:

Mouth-to-mouth ventilation

Mouth-to-nose ventilation

Mouth-to-mask ventilation

Resuscitator /mask ventilation

To ventilate ensure there is free access to the airway by using the head tilt and jaw thrust/ chin lift manoeuvre, see 3.1.

The movements of the chest can be clearly observed during insufflation and expiration. The insufflated volume is displayed directly on the monitoring instrument. The expired air can be felt and heard from the mouth and nose of the manikin with no risk of infection, as the air is the same as that insufflated by the trainee into the head bag, see 3.2. Accidental stomach inflation is simulated and can be observed in the stomach region and viewed directly on the monitoring instrument.

NOTE: Remember to change head bags and face pieces for each new trainee.

5.2 Compression ④

External chest compression can be practised, and the depth of compression will be displayed on the indicator bar.

5.3 Placement of AED pad

If the training with AED is performed AED pads can be placed on the manikin.

NOTE: The manikin can only be used with Training AED's

5.4. Removal of Face piece ⑤

Pull the ears outwards to release them from the two prongs on each side of the skull. Pull the mask upwards by the ears until free of skull, see 5.1.

5.5. Removal of Head bag ⑤

Grab the bag on each side, but avoid closing the opening. Pull bag slowly upwards, twisting from side to side if necessary, to facilitate removal, see 5.2.

6. Cleaning and disinfection

With the patented hygienic system of Ambu, incorporating exchangeable face piece and head bag, no internal cleaning and disinfection is necessary.

6.1. Head bag

The head bag is disposable and should always be discarded after training.

6.2. Face piece ⑥

The face piece may be reused after cleaning and disinfection. Remove the dental insert by pulling it from the face piece, see 6.1 and 6.2.

6.3. Manual cleaning

- A. Rinse face piece and dental insert in clean water.
- B. Wash items in warm water, max. 65 °C (150 °F), use a mild detergent.
- C. Rinse thoroughly in clean water to remove all traces of detergent.

6.4. Machine washing

The face pieces can be washed in an ordinary washing machine. Apply normal dose of detergent and choose a washing programme with maximum temperature of 70 °C (158 °F). In order to avoid the dental inserts rattling against the drum of the washing machine, the face pieces can be placed in a bag made of loosely woven fabric.

6.5. Disinfection

After separating and cleaning the face piece and dental insert, disinfection can be performed as follows:

- A. Place the items in a sodium hypochlorite solution with minimum 500 ppm freely accessible chlorine (1/4 a cup of liquid household bleach per gallon (approx. 4 litres) of tap water for 10 minutes). This solution must be fresh and should be discarded after use.
- B. Place the items in a 70 % alcohol chlorhexidine solution for 2 minutes (70 % Ethylalcohol and 0.5 % chlorhexidine). This method of disinfection is recommended by the Australian Resuscitation Council.
- C. Chemical disinfection can also be performed using recognised disinfectants suitable for use with polyvinylchloride (PVC). The supplier's directions on dosage and disinfection period should be closely followed.
- D. Always rinse the items in clean water after disinfection and allow to dry before storing.

NOTE: The face pieces must not be exposed to boiling, autoclaving or gas sterilisation.

6.6. Cleaning of skull, neck and body

The skull, neck and body of the manikin should be wiped over with a cloth moistened in a mild detergent and then wiped over once more using a cloth moistened in clean water. When washing, never allow detergent or water to enter the skull, body or instrument unit. If necessary, cover the connection between body and instrument unit with a cloth. Marks on the manikin left by lipstick or ball pen can penetrate the material and should therefore be removed as quickly as possible using alcohol.

6.7. Cleaning of clothing

The training suit is made of 50 % cotton and 50 % polyester and is washable at max. 40 °C (105 °F).

6.8. Assembly of face piece ⑦

Assemble the face piece by fitting the dental insert as shown and press it into the face piece so that only the flange and collar remain outside, see 7.1.

First, press the collar of the dental insert into the slot on one side of the face piece.

Next, take the edge and force it over the flange on the dental insert until the flange is seated in the slot in the mask all the way round, see 7.2.

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